

Intercall

antibacterial Pressure Mat



PM2 Pressure Mat

The Intercall PM2 pressure mat has been specifically designed to assist with fall prevention. Positioned alongside the bed or in a doorway, the unit will reliably alert staff when a resident is on the move without being intrusive. Manufactured from durable PVC, the **water resistant** properties of the PM2 pressure mat make it easy to wipe clean and prolong the life of the unit. The Intercall pressure mat has an antibacterial coating to assist with infection control and **features a non slip** backing allowing the unit to be used on all solid surfaces.



The **PM2 pressure mat** simply plugs into your existing Intercall call point and activates a standard call when pressure is applied to the mat. Supplied with a 350cm (11.6 feet) lead terminated with a moulded plug and measuring 91cm x 61cm (35 x 24 inch), the large surface area of the pressure mat ensures sensor activation and that the alarm will be raised.

features

- Antibacterial coating
- Natural non slip properties
- Attractive textured finish
- Manufactured from tough, durable PVC
- Water resistant and may be wiped clean
- Suitable for placing onto a wide variety of surfaces



Set up and test: Place the pressure mat onto a flat surface in the required position. Plug the PM2 lead into the jack socket on the front of the Intercall unit, step onto the mat and the red reassurance LED on the call point should flash. Step off the mat and press the reset button on the call point to cancel the alarm. This simple test must be carried out prior to every use of the mat. If at any time this mat does not function properly during testing, replace with a new mat and contact your Intercall supplier.

Safety Information: Do not bend the mat as this will cause internal damage and may prevent the unit from operating correctly. Do not place the mat on an uneven surface and always ensure the lead is routed carefully across the room to avoid hazards. The antibacterial coating has been tested with the following test bacteria: Klebsiella Pneumoniac and Staphylococcus Aureus. For high fall risk patients we recommend discussing your requirements with your local Intercall supplier.

If you would like further information on this or any other accessory available in the Intercall range, please contact your supplier who will be pleased to help.